
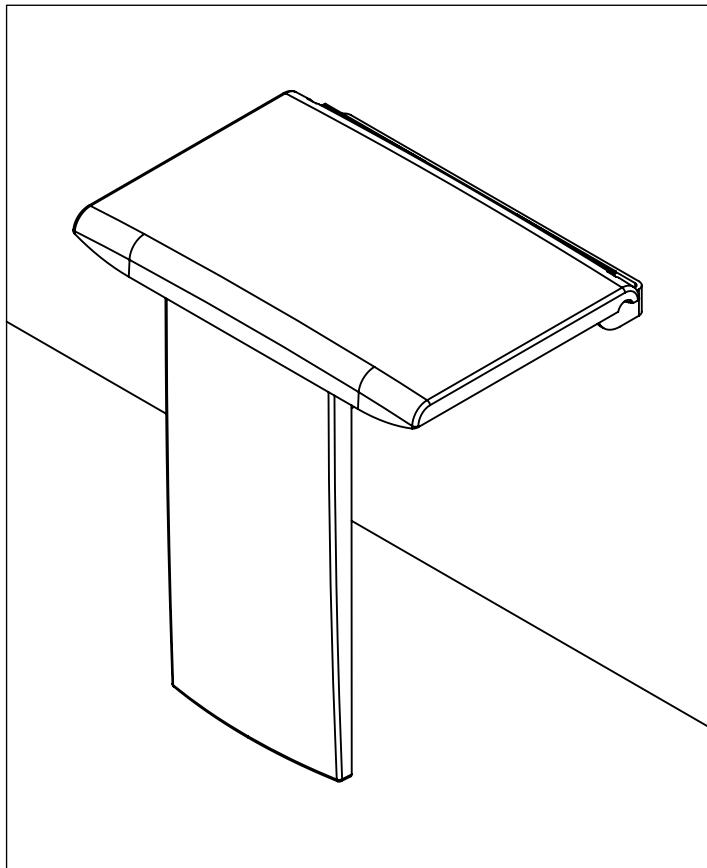
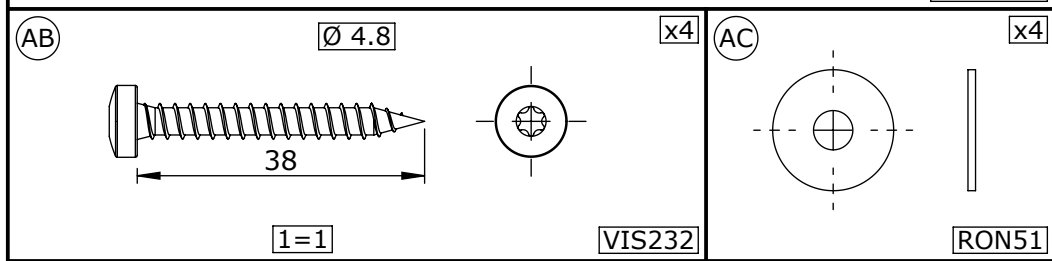
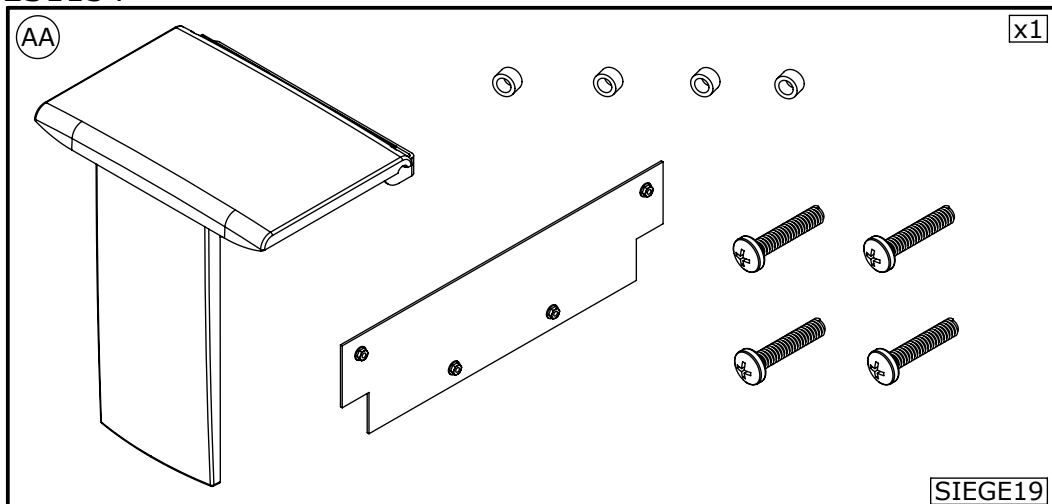
- F Notice de montage
- GB Installation instructions
- E Manual de instalación
- I Istruzioni di montaggio
- NL Montage-instructies
- D Montageanleitung

- Les quantités données sont les quantités utiles au montage. les sachets fournis contiennent des pièces supplémentaires.
- The quantities given are what are required for the installation. There may be some surplus in the package provide.
- Las cantidades indicadas son las cantidades necesarias paa el montaje. Las bolsitas incluidas contienen piezas suplementarias.
- Le quantità indicate sono quelle necessarie al montaggio. Sachetti forniti contengono dei pezzi più.
- De aangegeven hoeveelheid toont het aantal onderdelen dat voor montage nodig is. De meegeleverde zakjes kunnen leer onderdelen bevatten.

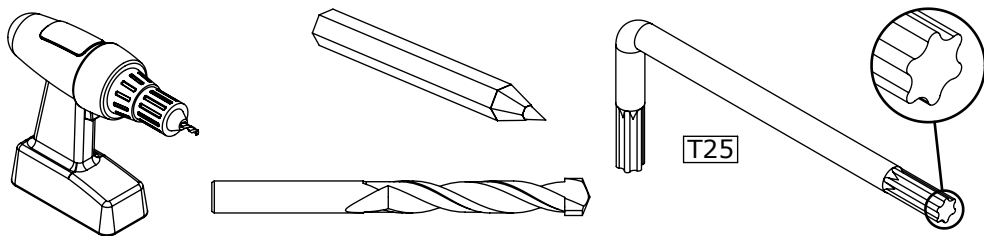F Matériel nécessaire **GB** Tools required **E** Material necesario
I Attrezzatura necessaria **NL** Nodige gereedschap

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GB Do not install this equipment before reading the following instructions :

Fitting instructions. Plugs have to be selected according to the type of wall.

The 4 fixing points have to be necessarily respected.

Care and maintenance. IMPORTANT : In order to guarantee the lasting quality of this product and its safety for the user , we recommand that when cleaning it you should use mild soap solutions. CAUTION : DO NOT USE cleaning products made of : etones, hydrocarbonrs, acids, bases, esters and ether. The disinfection has to be made according to the recommendations of use of the various products of disinfection. After use of a cleaning product, make a rinsing in the clear water.

Safety warnings. The weight supported by the seat is to 150kg in normal conditions of use (1 pers x sitting position) and subject to the conformity of the installation following the instructions. After assembly and before use, make sure the screws are tight. The manufacturer shall have no liability for any injury caused by improper installation or use of this seat.

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